



# Extremity Chamber

## QUICK SET-UP GUIDE

### Effective Treatment at Home




## PREPARATION AND SET-UP **No Smoking and No open flames!**

After your AOTI Representative has set up the equipment needed for use with this therapy.

### What you need: 1 Extremity Chamber Boot and 1 vial of Sterile water

**1** Ensure both the Concentrator and the Controller are plugged into power.

**2** When using Humidification: Twist off the top of the blue Sterile Water Vial and squeeze the water into the cup. Press and close the Humidifier cap. Plug in Humidifier cable into cup. Insert the Humidifier into the Boot facing toward the toe end of the boot. If you are **NOT** using Humidification, use the humidifier plug (silicone plug) to close the port.

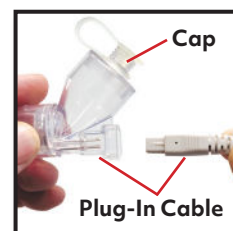


**3** Locate the two arrow connection points on the boot and hose. Line up the two arrows at the connection point to connect hose onto the boot. You will hear a click when it's connected.




### NOTE: VERY IMPORTANT

**4** Slip the leg into the Boot (pay attention to remove kinks, folds, etc.), and make sure the disposable boot cuff is below the knee in medium size and above the knee in a large size. **Note:** The cuff area **must have contact with skin to maintain a seal.**

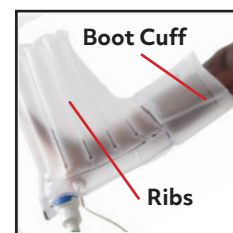

Switch on the Oxygen Concentrator

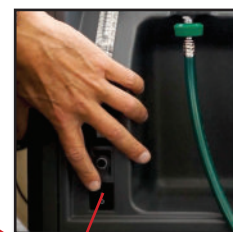

Switch On

## STARTING THERAPY

- 5** Turn the Controller (white machine) power **ON** by pressing the top most button.




- 6** The Display will prompt you to **INFLATE BAG**.

Press the bottom button (**ENTER**) for **YES**. The ribs of the Boot will inflate.



- 6** Inflate Bag? **Press Enter**

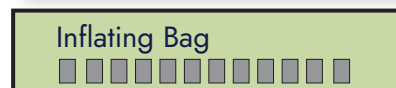


- 7** Display will prompt you to **INFLATE CUFF** (top of the Boot). Press the **ENTER** button for **YES**. **You will feel slight pressure at the cuff, which is normal.**

- 7** Inflate Cuff?



- 8** After the cuff is inflated, **START THERAPY** by pressing **ENTER** button for **YES**. Therapy will start, and pressure and time will be displayed. A typical therapy session lasts 90 minutes. Within 1-2 minutes, the P (pressure) number will start cycling between 10 mbar and 50\* mbar for the duration of the therapy. T (time) number indicates the number of minutes remaining in the therapy session.

- 8** Start Therapy?

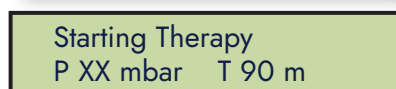

\*unless your physician has prescribed otherwise

## FINISHING THERAPY

- 9** **THERAPY FINISHED** alarm will sound and a light will flash. Press **ON/OFF** button to turn the Controller and the Oxygen Concentrator off.

- 9** Therapy Finished

- 10** Open the Cuff Plug to deflate the Cuff, and then remove the Boot.



- 11** Disconnect the Blue Interface Hose and **KEEP** the Humidifier and Humidifier Plug to reuse on the next Boot. Discard the Boot.



### REMINDER!

The Boot is single-use only. Discard after use. **DO NOT** reuse, reprocess or disinfect.

For a comprehensive manual and important safety information, refer to the INSTRUCTIONS FOR USE which is provided with the TWO2 Multi-Modality Therapy System. For further product information, contact a TWO2 Medical Representative. For medical questions, contact your doctor, nurse or caregiver. All trademarks designated herein are property of TWO2 and its licensors.

### WARNING



**NEVER SMOKE OR USE A NAKED FLAME IN THE PRESENCE OF OXYGEN.**

AOTI Contact: \_\_\_\_\_

Phone: \_\_\_\_\_ Email: \_\_\_\_\_



3512 Seagate Way, Suite 100  
Oceanside, CA 92056, USA  
P: 1-760-431-4700 • F: 1-760-683-3063

**Watch TWO2  
Quick Set-Up Video**



MB1081-D